



## Pulse Oximeter Quick Start Guide

Model # ADF-B06

---

### Installing the Batteries

**New batteries were installed in your device before shipping. Unless instructed by a support person, skip this section**

The pulse oximeter operates on 2 AAA batteries.



1. Open the battery cover on the back of the device.
2. Insert the batteries according to the "+" and "-" polarity symbols at the bottom of the compartment.
3. Replace the battery compartment cover.

**Do not allow other people to use your pulse oximeter. The pulse oximeter is specifically assigned to you and you only. Allowing other people to use your pulse oximeter can mislead your provider.**

### Taking a Measurement

1. Squeeze the closed end of the device to open the other end. Stick one finger completely into the rubber pad cavity of the Oximeter.
2. Keep the fingernail surface upward and release the closed end so the device fits snugly on your finger.
3. Press the power key to power on the Oximeter. If you do not completely insert your finger into the cavity, the measurement may be inaccurate.
4. The reading will automatically transfer to the RCP SmartHub and will then be sent to your provider.



#### Notes:

- Do not move your finger during the measurement.
- Keep your body still during the measurement.
- The Oximeter will automatically turn off 10 seconds after you remove your finger.

### Understanding the Results

- A normal %SpO<sub>2</sub> is 95% or greater.
- A normal PR is between 60-100.

